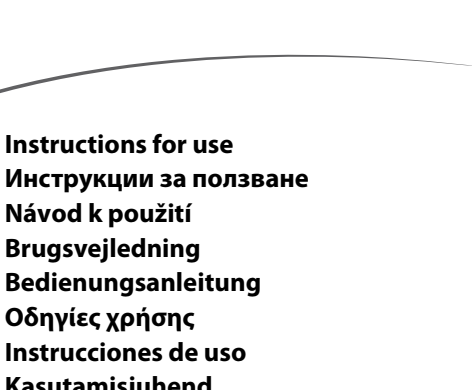
Ambu® Neuroline Concentric Neurology Needle Electrode

Instructions for use

Instructions for use

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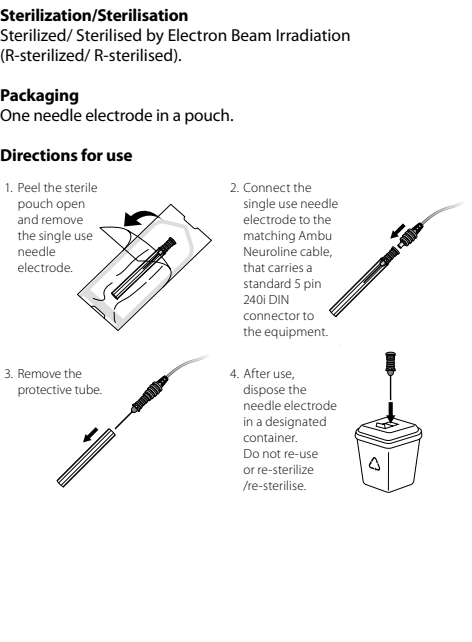
Neurology Needle Electrode

Intended Use

The Ambu® Neuroline Concentric Needle Electrode are made for muscle activity recording for Electromyography (EMG) applications. The needle electrodes are for single use only.

Precautions
<ol style="list-style-type: none">The Ambu® Neuroline Concentric Needle Electrode shall be used by health care professionals trained in performing neurophysiological examination. For single use only. Reuse of product can cause contamination, leading to infections. Do not soak, rinse or sterilize this device as these procedures may leave harmful residues or cause malfunction of the device. The design and material used are not compatible with conventional cleaning and sterilization procedures. Electrodes must only be used with Electromyography (EMG) applications monitoring devices in compliance with IEC 60601-1. During use, the electrodes should be placed as far as possible from the equipment which generates electromagnetic interference (EMI) to minimize unwanted radiofrequency interference (RFI). Do not use if individual pouch is damaged or has been opened. Rx only.
Sterilization/Sterilisation
Sterilized/ Sterilised by Electron Beam Irradiation (R-sterilized/ R-sterilised).
Packaging
One needle electrode in a pouch.
Directions for use

Ambu[®] is a registered trademark of Ambu A/S, Denmark
Ambu A/S is certified according to ISO 13485

Ambu [®] e registrirana trgovska marka na Ambu A/S, Daniia Ambu A/S e certifikovana v skladu s ISO 13485
Ambu [®] er et registreret varemærke Ambu A/S er ISO 13485-certificeret
Ambu [®] är en registrerad varemärke Ambu A/S är ISO 13485-certifierat
Ambu [®] er ein registrert varemærke Ambu A/S er ISO 13485-tilrettvirket
Ambu [®] ist eine eingetragene Marke der Ambu A/S, Dänemark Ambu A/S ist nach ISO 13485 zertifiziert.
Ambu [®] é uma marca registrada da Ambu A/S, Dania A Ambu A/S é uma companhia certificada de acordo com o padrão ISO 13485
Ambu [®] es una marca registrada de Ambu A/S (Dinamarca) Ambu A/S dispone de certificado según la norma ISO 13485
Ambu [®] is a registered trademark of Ambu A/S, Denmark Ambu A/S is certified according to ISO 13485
Ambu [®] er et registreret varemærke Ambu A/S er ISO 13485-certificeret
Ambu [®] är en registrerad varemärke Ambu A/S är ISO 13485-certifierat
Ambu [®] er ein registrert varemærke Ambu A/S er ISO 13485-tilrettvirket
Ambu [®] ist eine eingetragene Marke der Ambu A/S, Dänemark Ambu A/S ist nach ISO 13485 zertifiziert.
Ambu [®] é uma marca registrada da Ambu A/S, Dania A Ambu A/S é uma companhia certificada de acordo com o padrão ISO 13485
Ambu [®] es una marca registrada de Ambu A/S (Dinamarca) Ambu A/S dispone de certificado según la norma ISO 13485
Ambu [®] is a registered trademark of Ambu A/S, Denmark Ambu A/S is certified according to ISO 13485

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